


Amplification:

Before starting the amplification:

- ✓ **The extraction product must have a minimal DNA/RNA concentration of 5-10 ng/μl.**
- ✓ **The addition of the enzyme mix should be done with a pipette in the range of 1-2 μl.**
- ✓ **Keep the amplification tubes at 4°C at all times.**

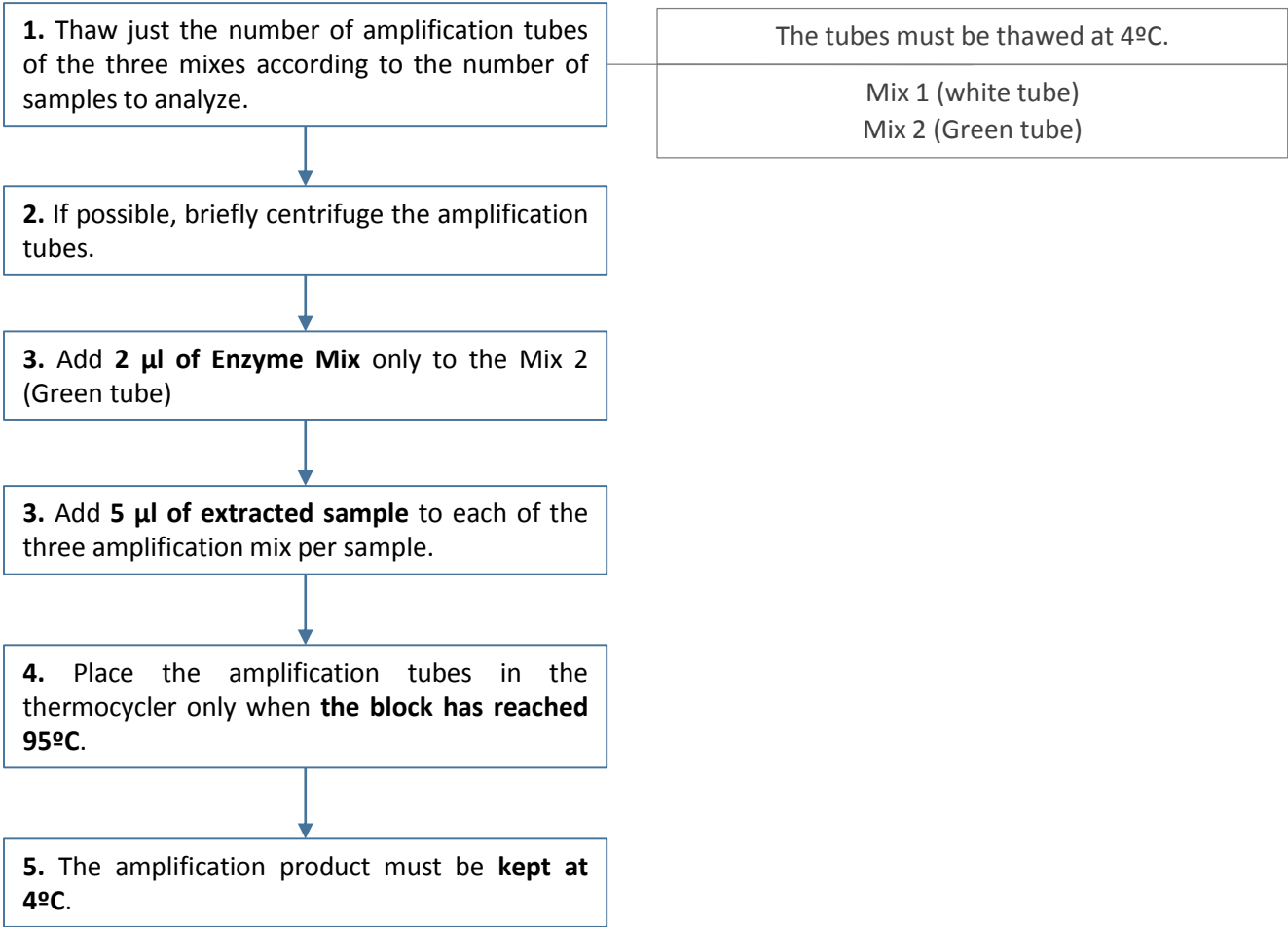

PCR program:

1 cycle	45°C	45'
	95°C	15'
45 cycles	95°C	30"
	56°C	1'30"
	72°C	1'
1 cycle	72°C	10'
1 cycle	4°C	∞